EcoVue® Sterile and Non-Sterile Ultrasound Gel Instructions for Use

Rx ONLY

MANUFACTURER:

HR Pharmaceuticals Inc.
2600 Eastern Boulevard, Suite 201
York, PA 17402 USA
(877)252-1110
www.ecovue.com

PRODUCTS

REF	Product Description	Sterile	Use	Package Size
280	EcoVue® Ultrasound Gel 20g Sterile SafeWrap™	Yes	Single-Use	20g
281	EcoVue® Ultrasound Gel 20g Non-Sterile Packet	No	Single-Use	20g
282	EcoVue® Ultrasound Gel 32g Sterile SafeWrap™	Yes	Single-Use	32g
283	EcoVue® Ultrasound Gel 32g Non-Sterile FlexPac™	No	Single-Use	32g
284	EcoVue® Ultrasound Gel 60g Non-Sterile Tube	No	Multi-Use	60g
285	EcoVue® Ultrasound Gel 250g Non-Sterile FlexPac™	No	Multi-Use	250g

INDICATIONS FOR USE

EcoVue® Sterile and Non-Sterile Ultrasound Gels are ultrasound couplants intended to be used during medical ultrasound procedures to couple sound waves between a patient and the medical imaging electronics. The gels are intended for use in diagnostic and therapeutic ultrasound procedures which require coupling gel or fluid.

EcoVue® Non-Sterile Ultrasound Gels are intended for non-invasive procedures which contact intact skin only.

DEFINITIONS

SafeWrap™—A sterile pouch in which the individual ultrasound gel packet is placed. *SafeWrap* is a registered trademark of HR Pharmaceuticals, Inc., its subsidiaries or divisions.

FlexPac™—A multi-use flexible container, that is similar to a bottle of ultrasound gel. *FlexPac* is a registered trademark of HR Pharmaceuticals, Inc., its subsidiaries or divisions.

DESCRIPTION

EcoVue® Sterile and Non-Sterile Ultrasound Gels, are non-irritating, non-sensitizing and water-soluble ultrasound couplants intended to contact uncompromised skin where a medical ultrasound procedure would be necessary for diagnostic purposes. In addition, EcoVue® Sterile Ultrasound Gel may be used on or near mucosal membranes via natural body orifices.

PRODUCT FEATURES

Water-Soluble

Non-Staining

Latex Free

Dye Free

Instrument Friendly

Sterile

BPOC Compliant

Kosher

Paraben & Propylene Glycol Free

Probe Friendly

CONTRAINDICATIONS

EcoVue® Sterile and Non-Sterile Ultrasound Gels should not be used in:

- Any invasive procedures in which a device is passed through the tissue, e.g. needle aspiration, needle localization, tissue biopsy, or other surgical procedures
- Any procedure where EcoVue® Sterile or Non-Sterile Ultrasound Gels have direct contact with breached or compromised skin, such as a fresh surgical site or an open wound
- Any procedure where EcoVue® Non-Sterile Ultrasound Gels may contact or come near mucosal membranes.

WARNINGS

For EcoVue® Sterile Ultrasound Gel, do not use product beyond labeled expiration date as sterility and performance cannot be guaranteed.

Re-use of the EcoVue Single-Use Product (REF #'s 280, 281, 282, 283) could result in cross-contamination, risks of infection, or adverse reaction for the patient.

If irritation occurs, discontinue use and consult a physician.

PRECAUTIONS

Do not use product if package is damaged or opened.

Do not use if tamper evidence seal on EcoVue® Ultrasound Gel 250g Non-Sterile FlexPac™ (REF 285) is missing or damaged.

Do not refill EcoVue® Ultrasound Gel 250g Non-Sterile FlexPac™ (REF 285).

Read the Instructions for Use and keep them in a safe place.

Use the product only in accordance with its intended use, see INDICATIONS FOR USE.

Use of an ultrasound gel for a task other than that for which it is intended could result in risk to the patient or one which provides an unsatisfactory performance.

Ultrasound gels should be handled by personnel completely familiar with their use

PRE-USE, HANDLING, AND INSPECTION

The product must be thoroughly inspected upon receipt and prior to use to assure packaging is not damaged or defective. If any product damage or leakage is identified, the product should be discarded.

INSTRUCTIONS FOR USE

❖ EcoVue® Ultrasound Gel 20g and 32g Sterile SafeWrap™ (REF 280 & 282)

Sanitize* exterior of SafeWrap™ pouch prior to introduction to sterile environment.

Remove the SafeWrap™ outer pouch and place the EcoVue® Ultrasound Sterile Gel packet on the sterile tray.

Apply an amount of EcoVue® directly to the patient or probe as needed.

Discard package after use.

❖ EcoVue® Ultrasound Gel 20g, 32g Non-Sterile Packet (REF 281 & 283)

Apply an amount of EcoVue® directly to the patient or probe as needed.

Discard package after use.

❖ EcoVue® Ultrasound Gel 250g Non-Sterile FlexPac™ and 60g Tube (REF 284 & 285)

Sanitize cap and tip prior to opening. Remove tamper-evident seal.

Flip cap to open.

Apply an amount of EcoVue® directly to the patient or probe as needed.

Once opened, discard after 30 days or according to internal healthcare facility procedures (whichever is sooner), even if not empty.

*An appropriate sanitizer is an isopropyl alcohol based sanitizer. Sanitizer IFU should be reviewed for appropriate sanitization duration and method.

STORAGE CONDITIONS

To preserve product performance, store away from direct sunlight.

Store in a clean, dry place at room temperature (15°-30°C or 59°-86°F).

GLOSSARY OF SYMBOLS

SYMBOL	MEANING
Rx ONLY	Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Catalog number
	Manufacturer
	Do Not Use if Opened or Damaged
	Do Not Re-Use/Single-Use Only
	Caution: See Warnings and Precautions
	Consult Instructions for Use
	Lot Number
	Sterilized by Gamma Irradiation
	Non-Sterile
	Not made with natural rubber latex
	Keep away from sunlight

	<p>Expiration Date</p>
	<p>Store between temperature limitations</p>
	<p>Authorized Representative in the European Community</p>
	<p>Complies with MDD 93/42/EEC (Sterile Product)</p>
	<p>Complies with MDD 93/42/EEC (Non-Sterile Product)</p>
	<p>Kosher Certified</p>
	<p>Halal Certified</p>
	<p>Made In USA</p>

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